HEINE Charging Case CC1



HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 234838 2021-08-30

V-200.00.330

Produktübersicht
Product overview
Vue d'ensemble du produit
Partes del producto
Informazioni generali sul prodotto

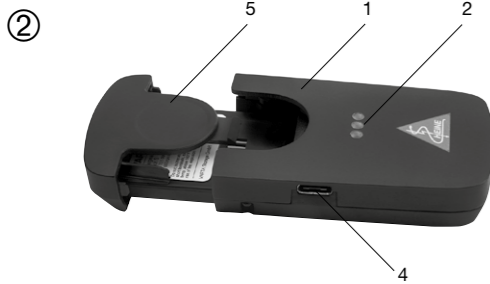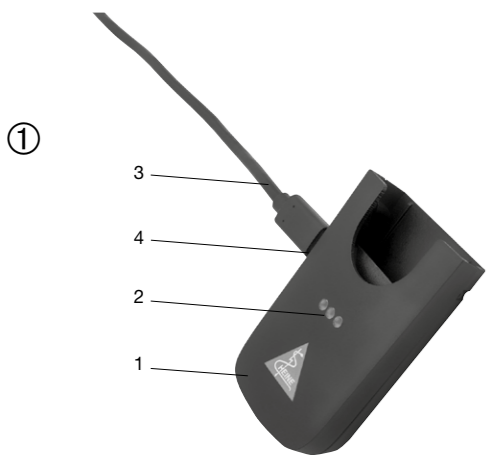HEINE Charging Case CC1

DEUTSCH

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HEINE Charging Case CC1

ENGLISH

Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE Charging Case CC1 is intended only for charging the rechargeable battery CB1 of the HEINE OMEGA 600 binocular indirect ophthalmoscope.

The product may only be used by qualified medical personnel and in professional health-care facilities.

! For U.S. only: Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

! CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

! NOTE! This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

Product overview (see fig. ①+②)

- Charging Case CC1
- Charge status indicator
- USB-C cable
- USB-C socket
- Rechargeable battery CB1

Setting up

For environmental reasons, we have decided to refrain from adding an additional power source. For charging, please use the power supply unit and the USB-C cable from your OMEGA 600.

The setting up and operation of the E4-USB is described in a separate instruction for use.

The Charging Case CC1 is intended for using in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see Fig. 3).

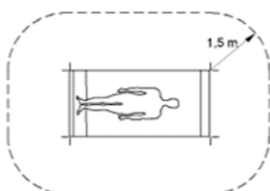
Fig.3

Operation

For charging, connect the USB power supply unit (e.g. E4-USB) to the USB-C socket (4) of the device. Insert the rechargeable battery CB1 (5) into the Charging Case CC1 (1). The charge status indicator (2) lights up briefly and the device is then automatically activated. During charging of the battery CB1 (5), the charge status indicator (2) is flashing.

Make sure that no foreign objects come between the battery CB1 (5) and the device. Make sure that the device is not contaminated in any form.

Removing and reinserting of the battery CB1 (5) will reset the system.

Hygienic reprocessing

The instruction is available at:

- www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient: All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform to the safety requirements from the IEC 60601-1 Medical electrical equipment.

Do not use power supplies where damage has been detected.

To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the switch mode power supply is accessible and disconnectable.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in presence of flammable gases / liquids, or in an oxygen rich environment.

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not use the device outdoors.

Heating during operation is normal and harmless.

Disposal

The Charging Case CC1 must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Dispose the rechargeable battery CB1 at your local collection point.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests	
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.	
Statement for the operational environments	Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
	The supply voltage quality should be that of a typical hospital environment.
Performance features of the ME system that have been determined to be essential to the performance	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
Note	Interruption of the charging process may occur due to a contact / air discharge. If so, please disconnect and reconnect the USB-C cable.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used as described in the chapter „Spare parts“. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B
Conducted emissions (EN 55011/CISPR 11)	Passed
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	
Immunity	See attached immunity test levels

Immunity test levels		
Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ±8 kV <p>Air Discharge: ±2 kV, ±4 kV, ±8 kV, ±15 kV</p>	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m <p>80–2700 MHz</p> 80 % AM at 1 kHz	
Electrical fast transients/ bursts (IEC 61000-4-4)	± 2 kV <p>100 kHz repetition frequency</p>	
Surges (IEC 61000-4-5)	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V <p>0.15 MHz – 80 MHz</p> 6 V in ISM bands between 0.15 MHz and 80 MHz <p>80 % AM at 1 kHz</p>	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m <p>450 MHz, FM: ±5 Hz deviation: 1 kHz sine; 28 V/m</p> 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m <p>810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m</p> 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m <p>2450 MHz; Pulse Modulation: 217 Hz; 28 V/m;</p> 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50Hz or 60 Hz	30 A/m; 60 Hz
Voltage dips (IEC 61000-4-11)	0 % U _r ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° <p>0 % U_r; 1 cycle and 70 % U_r; 25/30 cycles Single phase: at 0°</p>	
Short interruptions (IEC 61000-4-11)	0 % U _r ; 250/300 cycles	

Technical specification	
Environmental conditions for operation	+10 °C to +35 °C <p>30 % to 75 % rel. humidity</p> 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity</p> 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity</p> 500 hPa to 1060 hPa
Input	USB 2.0 Type C: 5 V, 1.2 A
Power consumption	max. 6 W
Protection class	Class II
Charging time	typ. 1.5 h

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Electromagnetic disturbances – Requirements and tests	
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.	
Statement for the operational environments	<p>Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p> <p>The supply voltage quality should be that of a typical hospital environment.</p> <p>Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.</p>
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	None
Warning	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p> <p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p> <p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</p>
Note	Interruption of the charging process may occur due to a contact / air discharge. If so, please disconnect and reconnect the USB-C cable.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used as described in the chapter „Spare parts“. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B
Conducted emissions (EN 55011/CISPR 11)	
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)	Passed
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	
Immunity	See attached immunity test levels

Immunity test levels		
Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ±8 kV <p>Air Discharge: ±2 kV, ±4 kV, ±8 kV, ±15 kV</p>	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m <p>80 –2700 MHz</p> <p>80 % AM at 1 kHz</p>	
Electrical fast transients/ bursts (IEC 61000-4-4)	± 2 kV <p>100 kHz repetition frequency</p>	
Surges (IEC 61000-4-5)	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V <p>0.15 MHz – 80 MHz</p> <p>6 V in ISM bands between 0.15 MHz and 80 MHz</p> <p>80 % AM at 1 kHz</p>	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m <p>450 MHz, FM: ±5 Hz deviation: 1 kHz sine; 28 V/m</p> <p>710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m</p> <p>810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m</p> <p>1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m</p> <p>2450 MHz; Pulse Modulation: 217 Hz; 28 V/m;</p> <p>5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m</p>	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50Hz or 60 Hz	30 A/m; 60 Hz
Voltage dips (IEC 61000-4-11)	0 % U _r ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° <p>0 % U_r; 1 cycle and 70 % U_r; 25/30 cycles Single phase: at 0°</p>	
Short interruptions (IEC 61000-4-11)	0 % U _r ; 250/300 cycles	

Environmental conditions for operation	+10 °C to +35 °C <p>30 % to 75 % rel. humidity</p> <p>700 hPa to 1060 hPa</p>
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity</p> <p>500 hPa to 1060 hPa</p>
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity</p> <p>500 hPa to 1060 hPa</p>
Input	USB 2.0 Type C: 5 V, 1.2 A
Power consumption	max. 6 W
Protection class	Class II
Charging time	typ. 1.5 h

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