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Please read and follow these instructions for use and keep them for future reference.

Intended Use

BETA X VET Otoscope

The HEINE BETA X VET Otoscope is intended for the transient examination of the ear, the external auditory canal and tympanum. The instrument has a main light which combines an optical magnifying equipment and an illumination equipment powered by a rechargeable battery.

The BETA X VET Otoscope can also be used for instrumentation in the ear with non-active medical devices.

The instrument has also the additional light HEINE inSPECT for non-contact, non invasive and transient examination of the body, for oral cavity and throat examination as well as anterior segment of the eye examination and the pupil reflex observation. The nose can only be examined transient with tips or specula.

The product may only be used by qualified veterinarians and in enclosed spaces. The application is limited to small animals (e.g. dogs, cats, domestic animals) and farm animals (e.g. cattle, pugs, sheep, goat, horse).

X CS1 VET Single and X CS1 VET Dual

The HEINE X CS1 VET is a charging station intended exclusively for recharging the internal power supply of the HEINE X-series instruments.

The product may only be used by qualified veterinarians or trained staff authorized to operate such devices and in enclosed spaces.

E4- USB and E12-USB

The HEINE E4-USB / E12-USB is intended for charging exclusively with the HEINE USB power supplies.

The product may only be used by qualified veterinarians or trained staff authorized to operate such devices and in enclosed spaces.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and safety information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent.
(Background color yellow; foreground color black)

NOTE! This symbol indicates valuable advice. These notes are important, but not related to hazardous situations.

Product overview

- | | | | |
|---|-----------------------------------|----|---|
| 1 | Slit | 7 | AllSpec tips (single use) |
| 2 | HEINE XCHANGE System SL-Magnifier | 8 | XV BETA X VET Specula |
| 3 | Additional light HEINE inSPECT | 9 | Charging compartment |
| 4 | X CHANGE System 3x Magnifier | 10 | Charge status indicator of the instrument |
| 5 | X CHANGE System 4.2x Magnifier | 11 | USB-C socket for charging the instrument |
| 6 | SANALON S tips (re-use) | 12 | USB-C socket on the bottom for setting up with the power supply |

Setting up

To start using the instrument, make sure that the X CHANGE system magnification attachment is properly locked into the bayonet mount. Before using the device for the first time, remove all protective films and fully charge the device.

Operation

The maximum temperature at the AllSpec tip, with an ambient temperature of 35°C and continuous operation, is 42,5 °C.

The BETA X VET Otoscope may only be inserted into the auditory canal when a tip / speculum is fitted.

Place the HEINE tip/speculum on the BETA X VET Otoscope so that the projection inside the tip fits into the bayonet slit (1). Tighten the tip by twisting it slightly to the right.

Do not use force when securing the tip/specula.

The different X CHANGE system magnifications can be exchanged as needed for the different tips/speculums.

Magnifier	Compatible tip/specula
HEINE XCHANGE System SL-Magnifier	SANALON S tips, AllSpec tips, XV BETA X VET Specula
X CHANGE System 3x Magnifier X CHANGE System 4.2x Magnifier	SANALON S tips, AllSpec tips

They are attached to the instrument via a bayonet mount (2) by turning it clockwise.

By pressing any button, you switch the device on. Use the Power Button  to switch the device off. This will immediately activate the otoscopy light at maximum brightness. The button  allows you to turn the otoscopy light on directly, and by pressing the same button again, you can decrease/increase the brightness.

The button  also activates the HEINE inSPECT additional lighting. Pressing the same button again will increase/decrease the brightness. When changing the light source, the previous light source will automatically be switched off, ensuring that only one light source is active at a time.

Button	Designation	Function	Application site
	HEINE inSPECT	Additional lighting: On/Min/Max	Minimum brightness level: anterior segment of the eye/pupil
			Maximum brightness level: skin, oral cavity & throat. Not suitable for eye examinations.
	Otoscopy F.O.	otoscopy lighting: On/Max/Min	Both levels: ear, ear canal and eardrum. Not suitable for eye examinations.
	ON/OFF	switch the device on and off	n/a


If no button is pressed for 5 minutes, the instrument will automatically switch off.

Hard reset:

Should a reset be necessary, it can be reset by pressing the on/off button for a period of 10 seconds.

Charging


Charge the device outside of the patient environment (1.5 m).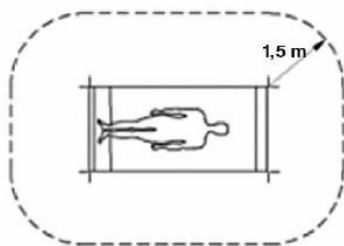

Charging via USB-C

The mains socket adapter of the power supply is interchangeable, this enables worldwide operation. The pictograms describe the procedure of changing the adapters:


Primary plug:

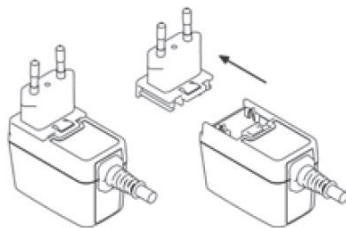EU

USA/Japan

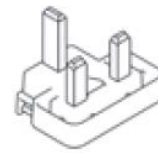UK

Australia

Connect the USB cable to the USB-C port of the otoscope or the X CS1 charger and the HEINE power supply.

Place the power supply in such a position where it is accessible and easily unplugged. The green LED on the E12-USB indicates the operational readiness of the power supply.

After you checked the mains voltage, connect the switch mode power supply to the mains supply.
When taking the instrument out of service, proceed in reverse order.

Charging via X CS1 Single/ Dual

The HEINE X CS1 charger is intended to be placed on a table or a comparable stable surface. In addition, the charger provides an M4 thread on the bottom. This can be used to mount the charger if required.

To set up the charger, a 5 V power supply (not included in delivery) is required, which meets the necessary safety requirements from the IEC 60601-1 Medical electrical equipment. The HEINE E12-USBC power supply is recommended, which can be purchased separately.

For setting up, connect the power supply to the USB-C port on the bottom of the charger (12). Make sure that the USB-C cable is firmly connected to the charger.

To ensure all-pole disconnection from the mains at all times, the device must be installed so that the switch mode power supply is accessible and disconnectable.

To put the charger out of operation, disconnect the charger from the power supply.

For charging the X Series instrument, place it completely in the charging compartment (9) of the charger as pictured in the product overview. Make sure that the charge status indicator of the instrument (10) is visible. Once the instrument has been correctly inserted into the charger, the instrument is automatically charged.

Make sure that there are no objects in the charger.

Make sure that only hygienic reprocessed X series instruments will be inserted into the charger.

The charge status indicator on the X series instrument (2) displays that it is being charged. It also displays the charge status. If the charge status indicator on the X Series instrument does not light up after placing the instrument in the charger, please check whether the power supply is plugged into the mains and the charger and that the instrument is correctly inserted into the charger. If necessary, reset the charging process by removing the instruments from the charger for a few seconds and then reinserting it. If you do not notice any improvement, please contact the manufacturer.

Battery indicator

Instrument	ChargerBattery	Charge	indicator segments		
			Green - 1	Green - 2	Orange - 3
switched off	connected	100 %	on	on	on
		full (99 % - 66 %)	flashing	on	on
		sufficiently charged (66 % - 33 %)	off	flashing	on
		empty (≤ 33 %)	off	off	flashing
switched on	not connected	full (≥ 66 %)	on	on	on
		sufficiently charged (66 % - 33 %)	off	on	on
		empty (33 % - 10 %)	off	off	on
		Residual capacity (≤ 10 %)	off	off	flashing

Hygienic reprocessing

BETA X VET Otoscope, magnifier, specula, XCS1 VET Single and X CS1 VET Dual

The instructions are available:

- online at www.heine.com

- in a paper version which you can request from the address listed

The HEINE AllSpec® disposable tip is a single-use product. Under no circumstances should it be reused, as this may cause severe mechanical damage.

E4-USB and E12-USB

Clean the cable and the power supply unit only if necessary and only with a dry cloth.

Maintenance

There is no regular maintenance required.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

BETA X VET otoscope and chargers: The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years.

Beyond this period, the product may continue to be used if it is in safe and good condition.

Speculum: The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 3 years. Beyond this period, the product may continue to be used if it is in safe and good condition.

Note to the user:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG.

The device only indicates the year of manufacture. The service life starts from the first of January of the year of manufacture.

Charge your device during longer storage periods to protect the battery from deep discharge.

General Warning

Otoscope: Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Charger: Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area.

Do not place metallic objects (e.g. paper clips, coins) in the charging compartment.

This product is not allowed to enter or be used in areas with strong magnetic fields, e.g. MRI scanners.

Do not modify the device.

Do not open the enclosure.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not shine the otoscopy light into the eyes. Ensure that the HEINE inSPECT additional light is activated at the minimal brightness level for this purpose.

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the safety requirements from the IEC 60601-1 Medical electrical equipment.

Do not use power supplies or cables if you notice any damage.

Check the tip before each use for rough surfaces and edges. Disposable tips (HEINE AllSpec | HEINE UniSpec) are for single use only. Do not use the device to examine injured tissue.

Do not use the device outdoors.

Use power adapter only in dry environment. Avoid splashing water. Never immerse it in liquids.

Heating during operation is normal and harmless. Make sure that the device isn't covered during operation, that it isn't placed in close proximity to a heating and that isn't exposed to direct sunlight.

The device must not be used with appliance outlets of extension leads.

The power supply is only approved for the use with ungrounded loads (Class II).

Use this power supply only for the supply of the devices it is intended for.

Warning: Danger to life due to electric shock:

- Only connect the power supply unit to an appropriate socket.
- Never carry or pull the power supply unit by the electrical cable.
- Only pull on the plug to disconnect a plug connection, never on the cable.
- Lay cables in a protected manner and ensure that nobody can fall over them.

Light exposure hazard

Prolonged intense light exposure can damage the retina, so the use of the device for eye examination should not be unnecessarily extended, and the brightness should not be set higher than necessary for clear observation of the target structure. The exposure dose for photochemical retinal hazard is the product of irradiance and exposure time. If the irradiance is reduced by half, the exposure time may be twice as long to reach the maximum limit.

Although no acute optical hazards have been identified with the HEINE BETA X VET otoscope, it is recommended that only the HEINE inSPECT lighting at the low setting be used for eye examinations. Using any lighting setting other than the HEINE inSPECT lighting at the low setting for eye examinations increases the risk of eye damage. The light source must be positioned at a distance of 100 mm or more from the patient's eye. The risk may also be increased if the patient being examined has already been examined with this or any other ophthalmic instrument within the last 24 hours, especially if the eye was exposed to a fundus camera.

ISO 15004-2:2007: Group 2 – inSPECT Additional Light at 100mm Working Distance

Warning – The light from this instrument may be harmful. The risk of eye damage increases with exposure time. Exposure with this instrument at maximum intensity for longer than 2 minutes exceeds the recommended threshold for hazard.

Since there is a significant risk of injury at exposures exceeding 10 J/cm², the user should avoid exposures longer than 2 minutes.

Disposal

The product must be recycled as separated electrical and electronic devices.
Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests	
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should ensure that it is used in such environments.	
Statement for the operational environments	Inside professional healthcare facilities except for near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.
	The supply voltage quality should be that of a typical hospital environment
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B
Harmonic Emissions	Passed*
Voltage fluctuations/Flickers	
Immunity	Passed*

*n/a: „Not applicable“ in the internally powered mode

Technical specifications**BETA X VET Oscope**

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+ 5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa Less than 1 month: -20 °C to +45 °C Less than 3 months: +0 °C to +30 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	- 20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	5 V
Nominal current	500 mA
Protection class	Internally powered Class II (charging)
Device classification according to IEC 62471	Risk Group 2
Cable	USB-cable approx 1,70m

XCS1 VET Single and X CS1 VET Dual

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+ 5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	- 20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
USB-C (input)	5 V; max. 1.2 A
USB-C (output)	5 V DC; max. 0.5 A per each output
Power consumption	≤ 15 W
Protection class	II (through external power supply)

E4-USB

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+ 5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	- 20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
Nominal input voltage	100 – 240 V AC / 50 – 60 Hz
Power consumption	< 15 W
Output voltage	5 V
Output current	max. 1.2 A
Protection class	II
Protection provided by enclosures (IP code)	IP20

E12-USB

Environmental conditions for operation	0 °C to +40 °C 0 % to 95 % rel. humidity up to 5000 m
Environmental conditions for storage	-40 °C to +80 °C 0 % to 95 % rel. humidity up to 5000 m
Environmental conditions for transport	-40 °C to +80 °C 0 % to 95 % rel. humidity up to 5000 m
Nominal input voltage	100 – 240 V AC / 50 – 60 Hz / 0.45 A
Power consumption	max. 15 W
Output voltage	5 V
Output current	3 A
Protection class	II
Protection provided by enclosures (IP code)	IP 20

Accessories

The corresponding accessories are listed at the following table:

AllSpec Disposable Tips - 2.5 mm (1x 250 pcs)	B-000.11.153
AllSpec Disposable Tips - 2.5 mm (1x 1000 pcs)	B-000.11.128
AllSpec Disposable Tips - 2.5 mm (10x 250 pcs)	B-000.11.151
AllSpec Disposable Tips - 4.0 mm (1x 250 pcs)	B-000.11.152
AllSpec Disposable Tips - 4.0 mm (1x 1000 pcs)	B-000.11.127
AllSpec Disposable Tips - 4.0 mm (10x 250 pcs)	B-000.11.150
Sanalon S Reusable Tip - 2.4 mm	B-000.11.107
Sanalon S Reusable Tip - 3.0 mm	B-000.11.108
Sanalon S Reusable Tip - 4.0 mm	B-000.11.109
Sanalon S Reusable Tip - 5.0 mm	B-000.11.110
Sanalon S Reusable Tips - 2.4 3 4 5 mm	B-000.11.111
XV4 Speculum	G-000.21.320
XV 6 Speculum	G-000.21.321
HEINE XCHANGE System SL-Magnifier	G-000.21.220
X CHANGE System Magnifier 3x	B-000.11.310
X CHANGE System Magnifier 4.2x	B-000.11.311
USB cord with E4-USB plug-in power supply	X-000.99.300
X CS1 Single	X-002.99.215
X CS1 Dual	X-002.99.216

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