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Instructions for use

ENGLISH

Rechargeable batteries HEINE Li-ion mPack/Li-ion mPack LL

Read these instructions carefully before using the rechargeable battery HEINE Li-ion mPack/Li-ion mPack LL and keep them in a safe place for future reference.

Intended use: The rechargeable battery HEINE Li-ion mPack/Li-ion mPack LL is for use with HEINE mPack and HEINE mPack LL only.

General safety instructions:

To ensure optimum performance and safety, please follow these instructions:

- Do not expose to fire
- Do not short circuit
- Do not modify or dismantle
- Do not drop or expose to excessive vibration
- Not permitted: Sterilisation, autoclaving, gas sterilisation and Sterrad
- Do not expose to direct sunlight
- Keep out of reach of children
- Store in a cool, dry place

Caution: Dispose of used batteries according to the instructions.

Caution: Risk of explosion if HEINE Li-ion mPack/Li-ion mPack LL is replaced by an incorrect type.

Safety in use:

The ideal ambient temperature during charging is between 10 °C and 35 °C. Do not recharge the battery Li-ion mPack/Li-ion mPack LL at ambient temperatures below 0 °C. This can destroy the rechargeable battery.

Setting up

- Detach the charging plug from the HEINE mPack or HEINE mPack LL.
- Unscrew the screw above the clip with a coin or screwdriver and remove the lid.
- The rechargeable battery HEINE Li-ion mPack/Li-ion mPack LL can be removed after detaching the cord. No tools are needed to do this.
- Insert the new battery so that the label is visible and the battery cord is directed towards the front control.
- Plug the plug of the battery into the socket on the circuit board.
- To close, locate the two pins on the lid in the cut-outs in the lower half of the housing. Close the lid and tighten the screw.
- Recharge the HEINE mPack or HEINE mPack LL for 2 hours just before use.

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