Instruction for use
HEINE E12-USB
power supply

Instruction for use HEINE E12-USB power supply

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

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HEINE E12-USB

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The HEINE E12-USB is intended for charging exclusively with the HEINE USB power supplies. The product may only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).

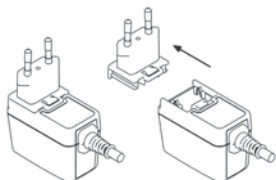
NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

E12-USB

Setting up

Primary plug:

EU

USA | Japan

UK

Australia

The mains socket adapter of the power supply is interchangeable, this enables worldwide operation. The pictograms describe the procedure of changing the adapters.

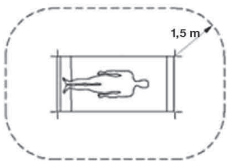
Connect the USB cable to the HEINE instrument.

Place the power supply in such a position where it is accessible and easily unplugged. The green LED on the power supply indicates the operational readiness of the power supply

After you checked the mains voltage, connect the switch mode power supply to the mains supply. When taking the instrument out of service, proceed in reverse order.

The charger is intended for installation in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see figure).

Hygienic Reprocessing

Clean the cable and the power supply unit only if necessary and only with a dry cloth.

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and | or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

⚠ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area.

Do not modify the device.

Do not open the enclosure.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not use the power supply outdoors.

The device must not be used with appliance outlets of extension leads.

The power supply is only approved for the use with ungrounded loads (Class II).

Before operating the device, verify that the mains voltage corresponds to the value indicated on the power supply.

Heating during operation is normal and harmless. Make sure that the device isn't covered during operation, that it isn't placed in close proximity to a heating and that isn't exposed to direct sunlight.

Use this power supply only for the supply of the devices it is intended for.

Warning: Danger to life due to electric shock:

- Only connect the power supply unit to an appropriate socket.
- Never carry or pull the power supply unit by the electrical cable.
- Only pull on the plug to disconnect a plug connection, never on the cable.
- Lay cables in a protected manner and ensure that nobody can fall over them.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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