
Instruction for use

HEINE BETA® 4 SLIM NT


HEINE BETA® SLIM

(a)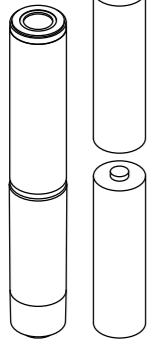

(b)

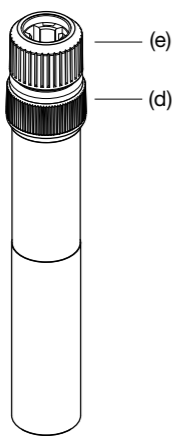(c)

HEINE BETA 4 SLIM NT / BETA SLIM

DEUTSCH

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HEINE BETA 4 SLIM NT / BETA SLIM

ENGLISH

Please read and follow these instructions for use of and keep them for future reference.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Intended Use

The HEINE BETA 4 SLIM NT / BETA SLIM handle is intended exclusively for operation of medical examination instruments manufactured by the HEINE company. The product may only be used by qualified medical professionals and in professional healthcare facilities.

Warnings and safety information

⚠️ Caution! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow, foreground color black)

⚠️ Note! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

HEINE handle variants

- ➊ BETA 4 SLIM NT Rechargeable handle X-007.99.474
- ➋ BETA SLIM Rechargeable handle X-002.99.471
- ➌ BETA SLIM Battery handle X-001.99.105

Setting up

Unscrew the bottom insert (a) from the handle shell and insert the rechargeable battery in direction of the arrow – or the battery leading with the positive terminal into the casing. Then screw the bottom insert (a) back in.

Only charge the rechargeable handle with the following HEINE chargers:

NT4: BETA 4 SLIM NT rechargeable handle

NT300: BETA SLIM rechargeable handle

Setup and operation of the HEINE chargers are described in separate instruction documents.

The automatic connector (AV)

Attaching the instrument heads

Attach the instrument head onto the handle from above in such a way that the connector fits into the guides on the handle head. Push the instrument into the handle head until it locks in place with an audible „click“.

Removing the instrument

To remove the instrument head turn the ring (e) to the right with the thumb and forefinger and, using the other hand, pull the instrument head up and out.

The twist-lock connector (TL)

Attaching the instrument heads

To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

Removing the instrument

To remove the head rotate it anti-clockwise and pull it off.

Operation

To switch the instrument on turn the rheostat (d) clockwise. The light intensity is indicated above the bar .

Position „1“ shows the maximum brightness; you can set any intermediate level as desired. To turn the device off, turn the rheostat back to position „0“.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Routinely inspect the condition of the battery and replace them if necessary.

Changing the batteries

(For implementation, see the relevant section)

Unscrew the bottom insert (a) from the handle shell (c) and replace the (rechargeable) battery (b).

Service

The device has no components serviceable by the end-user.

General notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you don't use the device for a longer period of time, please remove the batteries in advance.

Ensure that moisture does not penetrate the area around the rechargeable battery and the bottom insert.

Check the (rechargeable) batteries regularly. In case of any abnormalities in the operating or recharging times, replace the (rechargeable) batteries.

If the rechargeable battery has not been recharged for a long period it may discharge automatically.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners. Do not modify the device.

Repairs shall only be carried out by qualified persons.

Use only original HEINE parts, spare parts, accessories and power sources.

Turn the handle off during the charging process.

The handle is not designed for continuous use. The operating time of the handle depends on the maximum operating time of the examination head.

Precautions for HEINE rechargeable batteries

Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.

Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage).

The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Dispose at your local collection point.

The appendix contains following tables

- Accessories
- Technical specification, Technical specifications of the HEINE rechargeable batteries
- Handle variants and components,
- Explanation of the used symbols

Accessories	
NT4 table charger	X-002.99.494
NT300 table charger	X-002.99.495

Option	
Adapter set (two pieces) for BETA4 SLIM NT Rechargeable handle / F.O. 4 SLIM NT Laryngoscope handle and BETA SLIM Rechargeable handle / Small F.O. Laryngoscope handle	X-000.99.086

Technical specification HEINE BETA 4 SLIM NT / BETA SLIM	
Environmental conditions for operation	+10 °C to +35 °C 30 % to 90 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa

Technical specification HEINE Rechargeable battery		
	M3Z 4 NT Li-ion	M3Z NiMH
Environmental conditions	30 % to 75 % rel. humidity 700 hPa to 1060 hPa	
Operation and charging	+10 °C to +35 °C	
Storage	+5 °C to +45 °C	
Transport	-20 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa	
Cell type	Lithium-ion technology	Nickel-metal hydride
Rated voltage [V]	3.5	3.6
Capacity [mAh]	800	1000
Charging time NT4 [h]	typ. 2.75	–
Charging time NT300 [h]	–	typ. 4

Handle variants and components			
	Handle variants	Bottom insert	Rechargeable battery/battery
➊	BETA 4 SLIM NT Rechargeable handle X-007.99.474	X-000.99.110	X-007.99.380
➋	BETA SLIM Rechargeable handle X-002.99.471		X-002.99.106
➌	BETA SLIM Battery handle X-001.99.105		2 x LR6/AA*

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