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Please read and follow these instructions for use and keep them for future reference.

Intended use

The dermatoscope is a medical examination light with an internal power supply for non-invasive transient contact and non-contact examination of the skin as well as for trichoscopic examination. Skin changes can be examined. The examination of injured skin is only permitted in non-contact mode. The device also has additional lighting for non-contact, non-invasive, and transient examination of the large areas of skin.

The product may only be used by qualified medical professionals and in professional healthcare facilities. The application is not limited to any patient demography.

Warnings and Safety Information

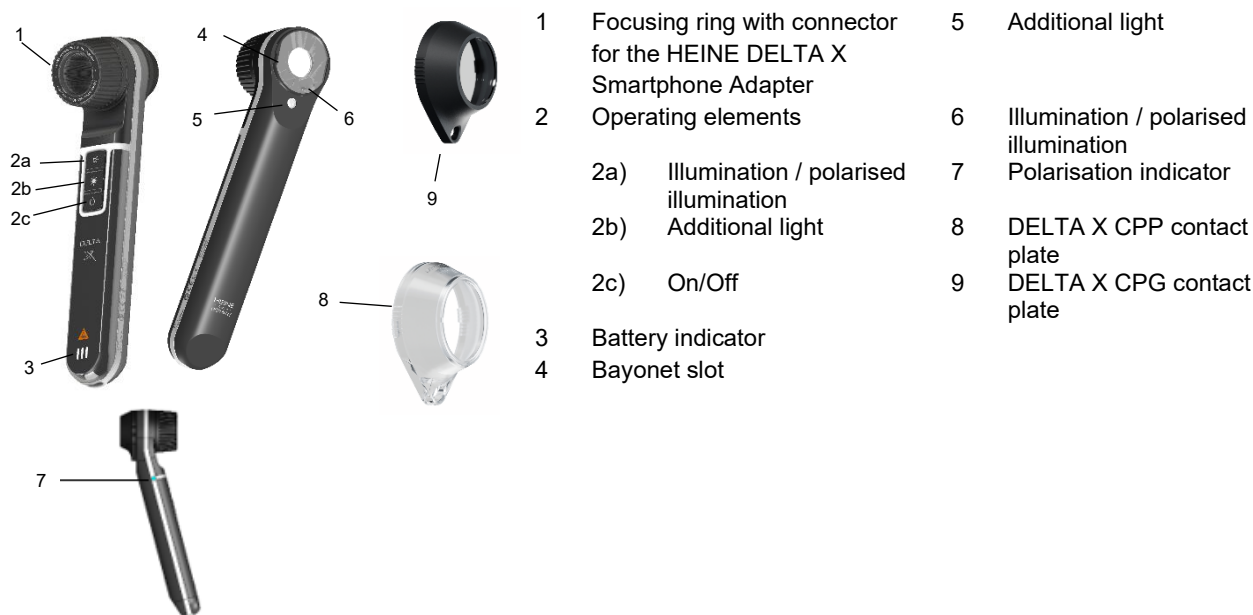
CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black)

NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview


- | | | | |
|-----|---|---|---------------------------------------|
| 1 | Focusing ring with connector for the HEINE DELTA X Smartphone Adapter | 5 | Additional light |
| 2 | Operating elements | 6 | Illumination / polarised illumination |
| 2a) | Illumination / polarised illumination | 7 | Polarisation indicator |
| 2b) | Additional light | 8 | DELTA X CPP contact plate |
| 2c) | On/Off | 9 | DELTA X CPG contact plate |
| 3 | Battery indicator | | |
| 4 | Bayonet slot | | |

Setting up

Charge the device fully before first use.

Charging

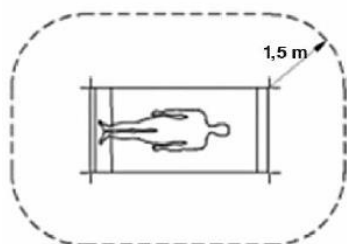Charge the device outside of the patient environment (1.5 m).

Charging via USB-C

Use the equipment provided with the DELTA X Dermatoscope. Connect the USB cable to the USB-C port of the dermatoscope and the HEINE power supply.

Using the charging station

For charging, insert the DELTA X into the X CS1 charging station. The charge indicator is located on the DELTA X.

Pay attention to the fact that no foreign objects are between the device and the charging station.

When using the DELTA X Dermatoscope, make sure that the charging station and the DELTA X Dermatoscope are not contaminated.

The setting up and operation of the charging station is described in a separate instruction of use.

Battery indicator

A charge indicator is located on the handle side of the dermatoscope (3).

Battery indicator (3)

Device	Charger	Battery capacity	Charge indicator segments		
			Green - 1	Green - 2	Orange - 3
switched off	connected/ inserted	100 %	on	on	on
		full (99 % - 66 %)	flashing	on	on
		sufficiently charged (66 % - 33 %)	off	flashing	on
		empty (≤ 33 %)	off	off	flashing
switched on	not connected	full (≥ 66 %)	on	on	on
		sufficiently charged (66 % - 33 %)	off	on	on
		empty (33 % - 10 %)	off	off	on
		residual capacity (≤ 10 %)	off	off	flashing

Operation

To switch on the illumination, press the button (2c) on the device. The device starts in polarisation mode. Alternatively, the dermatoscope can be switched on immediately with polarised illumination by pressing the button (2a).

Use the button (2b) to switch on the additional light.

The button (2c) is only used for switching off the device. Pressing this button switches off the dermatoscopic light or the additional light and the device.

If no button is pressed for 3 minutes, the instrument will automatically switch off.

Contact mode (with contact plate)

Place the contact plate (8, 9) on the device, making sure that it snaps into place. Please note that the cutout in the contact plate at the outer frame aligns with the additional light. The cutout is designed to allow the additional light to shine through.

Place the device with the contact plate onto the area of skin being examined. In polarised mode, the polarisation indicator (7) lights up blue. Use the polarisation button (2a) to switch to non-polarised mode. In this case, use a contact fluid such as immersion oil.

An unused DELTA X CCP contact plate must be used to examine the mucous membranes. This contact plate must be disposed after use. Under no circumstances should a DELTA X CPG contact plate be used for this use.

Do not use the contact plate, which is damaged.

Adjusting the focus in contact mode and non-contact mode

Your eye needs to be at a distance of approximately 2 cm from the focusing ring (1). Rotate the focusing ring until you see a sharp image. The image has 10-fold magnification.

Non-contact mode (without contact plate)

Take off the contact plate (8, 9) and hold the device at a distance of approximately 1 cm above the area of skin being examined. Use the polarisation button (2c) to switch to non-polarised mode.

Additional light in non-contact mode (with or without contact plate)

The additional light allows the illumination of the skin surface with and without the attached contact plate (8, 9). To operate, press button (2b).

Reset of the device

If the device cannot be operated through the battery is sufficiently charged, press button (2a) and (2c) for 8 seconds.

Option „Adapter Smartphone“

The "Smartphone Adapter" option can be used to mechanically connect a smartphone with a respective app. The app is designed for documentation by recording, storing and displaying images.

To connect the dermatoscope to a smartphone, you need a suitable adapter for your smartphone. Place the adapter onto the focusing ring (1). The adapter is held in place by magnetic force.

Install the corresponding app on your smartphone. The app is available in the App Store*. To avoid any possible malfunctions, please deactivate all other services (switch on flight mode).

*App Store is a service mark of Apple Inc.

Protecting patient data is users' sole responsibility (e. g. protection of the smartphone with a pin, data transmission via a secure connection). Ensure that your smartphone is adequately protected against access by third parties.

An electrical conducting connection between smartphone, PC and a mains power source is not permissible during the recording of the picture.

All images are saved exclusively on the smartphone in the app's sandbox. The user is responsible for the regular creation of a data backup to prevent data loss, which can happen for example through deleting the app or losing the smartphone. A data backup can be made via cable to another computer or via the network to the iCloud.

Hygienic Reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

There is no regular maintenance required.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

The device indicates the year of manufacturer. The service life starts from the first of January of the year of manufacturer.

Charge your device during longer storage periods to protect the battery from deep discharge.

Store and use the device only in a dry and dust-free environment.

The LOT number on the packaging refers to the contact plate.

After moving the device from a cold to a warm environment, allow sufficient time for it to acclimatize.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or flashing light or unusually bright or dimmed lightning.

Do not use the device in fire- or explosive risk area.

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the safety requirements from the IEC 60601-1 Medical electrical equipment.

Do not use power supplies or cables if you notice any damage.

Do not look directly into the light source to avoid dazzle from the intense light.

Eye examinations are not allowed with the dermatoscope.

Inform the patient that they must close their eyes during a facial examination.

Lay cables in a protected manner and ensure that no one can trip over them.

After opening the packaging, store the DELTA X CPP contact plate properly and protect it from contamination.

For separating the connection always pull on the plug, never on the cable.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside professional healthcare facilities except for near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.
	The supply voltage quality should be that of a typical hospital environment
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B
Harmonic Emissions	Passed*
Voltage fluctuations/Flickers	
Immunity	See attached immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ± 8 kV Air Discharge: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80-2700 MHz 80 % AM at 1kHz	
Electrical fast transients / bursts (IEC 61000-4-4)*	± 2 kV 100 kHz repetition frequency	
Surges (IEC 61000-4-5)*	$\pm 0,5$ kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)*	3 V 0,15 MHz – 80 MHz 6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ± 5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Proximity magnetic fields (IEC 61000-4-39)	134,2 kHz; PM 2,1 kHz; 65 A/m 13,56 MHz; PM 50 kHz; 7,5 A/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50 Hz or 60 Hz	
Voltage dips (IEC 61000-4-11)*	0 % U _T ; 0,5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°	
Short interruptions (IEC 61000-4-11)*	0 % U _T ; 250/300 cycle	

*n/a: „Not applicable“ in the internally powered mode

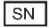
Accessories



The corresponding accessories are listed at the following link: www.heine.com/DELTA-X-dermatoscope-accessories-IFU

Technical specifications

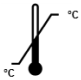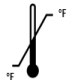Environmental conditions for operation	+10 °C to +35 °C 10 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+ 5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa Less than 1 month: -20 °C to +45 °C Less than 3 months: +0 °C to +30 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	- 20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
Nominal Voltage	5V
Nominal current	500 mA
Protection class	Internally powered Class II (charging)
Device classification according to IEC 62471	Risk group 1
Applied part	Contact plate Type BF
Charger and Cable	HEINE E4 USB C: USB cable approx. 2,0m Input: 100 – 240 V AC; 50 – 60 Hz Output: 5V; max. 1.2A HEINE E12 USB C: USB cable approx. 1,7m Input: 100 – 240 V AC; 50 – 60 Hz Output: 5V; max. 3.0A

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Symbolforklaring
Käytettyjen symbolien selitys
Explicação dos símbolos utilizados

	<p>CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukteverordnung (EU) 2017/745. The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745. Le marquage CE indique que le produit est conforme ou au règlement (UE) 2017/745 relatif aux dispositifs médicaux. La marca CE indica que el producto cumple el Reglamento europeo sobre productos sanitarios (UE) 2017/745. Il marchio CE indica la conformità con il regolamento sui dispositivi medici (UE) 2017/745. CE-märkning markerar en överensstämmelse med förordningen om medicinska produkter (EU) 2017/745. CE-markering duidt de overeenstemming aan met de verordening betreffende medische hulpmiddelen (EU) 2017/745. CE-mærkningen angiver overensstemmelse med forordningen om medicinsk udstyr (EU) 2017/745. CE-merket angir at produktet er i samsvar med forskriften om medisinsk utstyr (EU) 2017/745. CE-merkintä tarkoittaa, että laite lääkinnällisiä laitteita koskevan asetuksen (EU) 2017/745 kanssa. O símbolo CE identifica a concordância com o regulamento de Dispositivos Médicos (EU) 2017/74.</p>
	<p>Katalog- oder Bestellnummer Catalogue- or order number Numéro de catalogue ou de commande Número de catálogo o de pedido Codice catalogo e di dell'ordine numero Katalog- eller Beställningsnummer Catalogus- of Bestelnummer Katalog- eller Ordrenummer Katalog- eller bestillingsnummer Luettelo- tai viitenumero Número de catálogo ou pedido</p>
	<p>Seriennummer Serial number Numéro de série Número de serie Numero di serie Seriennummer Seriennummer Seriennummer Seriennummer Seriennummer Sarjanumero Número de série</p>
	<p>Hersteller Manufacturer Fabricant Fabricante Produttore Tillverkare Fabrikant Producent Produsent Valmistaja Fabricante</p>

	<p>Hersteldatum (Bezieht sich auf den ersten Januar des Herstelljahres) Date of manufacture (Refers to the first January of the year of manufacture) Date de fabrication (Se réfère au premier janvier de l'année de fabrication) Fecha de fabricación (Se refiere al primer día de enero del año de fabricación) Data di produzione (Si riferisce al primo gennaio dell'anno di produzione) Tillverkningsdatum (Avser den första januari tillverkningsåret) Productiedatum (Verwijst naar de eerste januari van het productiejaar) Produktionsdato (Henviser til den første januar i produktionsåret) Produksjonsdato (Gjelder den første januar i produksjonsåret) Valmistuspäivä (Viittaa valmistusvuoden tammikuun ensimmäiseen päivään) Data de fabricação (Refere-se ao primeiro de janeiro do ano de fabricação)</p>
	<p>Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie) Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE) Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE) Desechado separado de aparatos eléctricos y electrónicos (Directiva Europea RAEE) Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE) Separat insamling av elektriska och elektroniska apparater (det europeiska WEEE-direktivet). Gescheiden inzameling van elektrische en elektronische apparaten (Europese AEEA richtlijn) Produkter med dette symbolet skal ikke avhendes sammen med vanlig husholdningsavfall, men krever separat kassering i henhold til lokale bestemmelser. (European Waste Electrical and Electronic Equipment Directive, WEEE) Separat indsamling af elektrisk og elektronisk udstyr (det europæiske WEEE-direktiv) Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste (eurooppalainen WEEE-standardi). Coleção separada de aparelhos elétricos e eletrônicos (Diretrizes Europeias WEEE)</p>
	<p>Zulässiger Temperaturbereich in °C für Lagerung und Transport Temperature limits in °C for storage and transport Plage de température admise en °C pour le stockage et le transport Rango de temperatura permitida en °C para almacenar y transportar el producto Temperatura ammessa in °C per conservazione e trasporto Tillåtet temperaturintervall i °C för lagring och transport Toegestane temperaturen in °C voor opslag en transport Tilladt temperaturområde i °C ved opbevaring og transport Temperaturbegrensning i °C for oppbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C) Limite de temperatura permitida em °C para armazenamento e transporte</p>
	<p>Zulässiger Temperaturbereich in °F für Lagerung und Transport Temperature limits in °F for storage and transport Plage de température admise en °F pour le stockage et le transport Rango de temperatura permitida en °F para almacenar y transportar el producto Temperatura ammessa in °F per conservazione e trasporto Tillåtet temperaturintervall i °F för lagring och transport Toegestane temperaturen in °F voor opslag en transport Tilladt temperaturområde i °F ved opbevaring og transport Temperaturbegrensning i °F for oppbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F) Limite de temperatura permitida em °F para armazenamento e transporte</p>
	<p>Zulässige Luftfeuchtigkeit für Lagerung und Transport Humidity limitation for storage and transport Humidité admise pour le stockage et le transport Humedad del aire permitida para almacenar y transportar el producto Umidità atmosferica ammessa durante il trasporto e la conservazione Tillåten luftfuktighet för transport och lagring Toegestane luchtvochtigheid voor opslag en transport Tilladt luftfugtighed ved opbevaring og transport Fuktighetsbegrensning for oppbevaring og transport. Sallittu ilmankosteus kuljetuksen ja varastoinnin aikana Umidade do ar admissível para o armazenamento e transporte</p>

	<p>Zulässiger Luftdruck für Lagerung und Transport Pressure limitation for storage and transport Pression atmosphérique admise pendant le transport et le stockage Presión de aire permitida para almacenar y transportar el producto Pressione atmosferica ammessa durante il trasporto e la conservazione Tillåten lufttryck för lagring och transport Toegestane luchtdruk voor opslag en transport Tilladt lufttryk ved opbevaring og transport Trykkbegrensning for oppbevaring og transport Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana Pressão do ar admissível para o armazenamento e transporte</p>
	<p>Vorsicht Bruchgefahr! Fragile, handle with care! Fragile ! Manipuler avec soin Atención. Frágil. Attenzione: pericolo di rottura! Försiktig! Risk för brott Voorzichtig, kans op breuk! Forsiktig, risiko for brud! Ømtålig, behandles forsiktig! Varo särkymisvaaraa! Frágil, manuseie com cuidado!</p>
	<p>Trocken lagern Keep dry! Conserver au sec! Conservar en un lugar seco! Evitare ambienti umidi! Förvaras torrt! Droog bewaren! Opbevares tørt! Hold tørt! Säilytetään kuivassa paikassa! Armazenar em ambiente seco!</p>
	<p>Gebrauchsanweisung befolgen. (Hintergrund: blau, Vordergrund: weiß.) Follow instructions for use! (Background color: blue, foreground color: white.) Suivre le mode d'emploi. (Couleur de fond : bleu ; couleur du premier plan : blanc) Seguir obligatoriamente las instrucciones de uso. (Color de fondo: azul, color de primer plano: blanco) Attenersi obbligatoriamente alle istruzioni per l'uso. (Colore dello sfondo: blu, colore in primo piano: bianco) Bruksanvisningen ska alltid följas. (Bakgrundsfärg: blå, förgrundsfärg: vit) De gebruiksaanwijzing is bindend en dient gevolgd te worden. (achtergrondkleur: blauw, voorgrondkleur: wit) Følg altid brugsanvisningen. (Baggrundsfarve: Blå; forgrundsfarve: Hvid) Følg bruksanvisningen! (Bakgrunnsfarge: blå, forgrunnsfarge: hvit.) Käyttöohjetta on noudatettava tarkasti. (Taustaväri: sininen, etualan väri: valkoinen) Siga as instruções de uso! (Cor de fundo: azul, cor de primeiro plano: branco)</p>
	<p>Anwendungsteil des Typs BF Type BF applied part Partie appliquée de type BF Pieza de aplicación del tipo BF Applicazione di tipo BF Användningsdel för typ BF Gebruiksonderdeel van het type BF Anvendelsesdel type BF Anvendt del type BF Tyypin BF liityntäosa Parte de aplicação do tipo BF</p>

	<p>Bei beschädigter Verpackung nicht verwenden Do not use if package is damaged Ne pas utiliser si l'emballage est endommagé No utilizar si el envase está dañado Non utilizzare se la confezione è danneggiata Använd inte om förpackningen är skadad Niet gebruiken bij beschadigde verpakking Må ikke bruges, hvis emballagen er beskadiget Må ikke brukes hvis emballasjen er skadet Älä käyttää, jos pakkaus on vahingoittunut Não utilize se a embalagem estiver danificada</p>
	<p>Polarisation Polarisation Polarisation Polarización Polarizzazione Polarisation Polarisatie Polarisering Polarisasjon Polarisaatio Polarização</p>
	<p>Zusatzleuchte Additional light Éclairage supplémentaire Luz adicional Lampada supplementare Extralampa Extra lichtbron Ekstralampe Tilleggslampe Lisävalo Luz auxiliar</p>
	<p>Ein-/ Aus On/Off Marche/Arrêt On/Off ON/OFF Till/från Aan/uit Tænd/sluk På/av Päälle/pois Ligar/desligar</p>
	<p>Papier Paper Papier Papel Carta Papper Papier Papir Papir Paperi Papel</p>
	<p>Unique Device Identification</p>
	<p>Medical Device</p>

 **HEINE Optotechnik GmbH & Co. KG**

Dornierstr. 6, 82205 Gilching, Germany
www.heine.com

med 237222 2026-05-20