**Instruction for use**  
**HEINE handles**

# Instruction for use HEINE handles

|                   |           |
|-------------------|-----------|
| <b>DEUTSCH</b>    | <b>4</b>  |
| <b>ENGLISH</b>    | <b>8</b>  |
| <b>FRANÇAIS</b>   | <b>12</b> |
| <b>ESPAÑOL</b>    | <b>16</b> |
| <b>ITALIANO</b>   | <b>20</b> |
| <b>SVENSKA</b>    | <b>24</b> |
| <b>NEDERLANDS</b> | <b>28</b> |
| <b>DANSK</b>      | <b>32</b> |
| <b>NORSK</b>      | <b>36</b> |
| <b>SUOMI</b>      | <b>40</b> |
| <b>PORTUGUÊS</b>  | <b>44</b> |

## HEINE handle variants

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# HEINE Handles

This instruction document is valid for the following products from the HEINE handle series: BETA battery handle, rechargeable handle: BETA NT, BETA 4 NT, BETA 4 USB, BETA L, large battery handle, mini3000 battery handle and mini3000 rechargeable handle

Please read and follow these instructions for use of and keep them for future reference.

## Intended Use

The HEINE handles are designed exclusively for the operational use of medical examination instruments manufactured by the HEINE company. The product may only be used by qualified medical professionals and in professional healthcare facilities.

**For U.S. only:**

**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

## Warnings and Safety Information

**CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

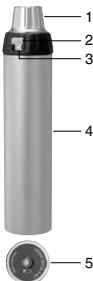
**NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

## Product overview

- 1 AV-/TL-Connector
- 2 Unlock button
- 3 Rheostat
- 4 Outer sleeve
- 5 Bottom insert
- 6 Switch
- 7 Clip

### BETA handles

### mini3000 handles

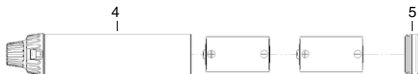
## Setting up

### Inserting battery and rechargeable battery

#### HEINE battery handle

To set up the HEINE battery handle, unscrew the bottom insert (5) from the handle module and insert the batteries with the positive terminal first into the outer sleeve (4). Screw the bottom insert (5) back in.

#### HEINE rechargeable handle

To set up the HEINE rechargeable handle, unscrew the bottom insert (5) from the handle module and insert the rechargeable battery into the outer sleeve (4) as described in the rechargeable battery instructions. Screw the bottom insert back in.

Only charge the rechargeable handle with the following HEINE chargers:

NT300: BETA NT and BETA L rechargeable handle

miniNT: mini3000 rechargeable handle

NT4: BETA 4 NT rechargeable handle

E4-USB-power adapter: BETA 4 USB rechargeable handle

Setup and operation of the HEINE chargers are described in separate instruction documents.

### Charging of the Li-ion L battery

The BETA L, BETA 4 NT and BETA 4 USB handles are designed specifically for the use of the Li-ion L rechargeable battery. To check the battery charge, the bottom inserts are equipped with a charge level indicator.

- BETA L: The charge level indicator is automatically activated when the instrument is disconnected from the charging device. With the BETA L, the display activates when it is turned on and goes off automatically after 10 seconds.

Charge status indicators:

Orange/green/green: 66 – 100 %

Orange/green: 33 – 66 %

Orange: 10 – 33 %

Orange flashing: < 10 %

- BETA 4 NT: The charge level indicator shows a constant orange when the battery needs to be recharged. With a fully discharged battery, the charge level indicator goes out and the instrument head is disabled.

- BETA 4 USB: While charging, the charge level indicator flashes green. After charging is complete, the battery indicator turns constant green. The charge level indicator shows a constant orange when the battery needs charging. With a fully discharged battery, the battery level indicator goes out and the instrument head is disabled.

### Attaching the instrument heads

#### HEINE BETA handle and large battery handle

*Automatic Connector (AV)*

Attach the instrument head onto the handle from above in such a way that the connector fits into the guides on the handle head. Push the instrument into the handle head until it locks in place with an audible "click".

To remove the instrument head turn the connector (1) on the handle head to the right with the thumb and forefinger and, using the other hand, pull the instrument head up and out.

*Twist-Lock Connector (TL)*

To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

*Removing the instrument*

To remove the head rotate it anti-clockwise and pull it off.

#### HEINE mini3000 handle

The instrument head can be screwed on and off using the internal thread in the handle head.

### Operation

#### HEINE BETA handles and large battery handle

To activate the instruments press the unlock button (2) and turn the rheostat (3) clockwise. The light intensity is indicated by the bar .

Position "1" shows the maximum brightness; you can set any intermediate level as desired. To turn the device off, turn the rheostat back to position „0“.

#### HEINE mini3000 handles

To turn the instrument on, push the switch (6) down; to turn it off, push it back up.

When the clip (7) is clipped onto a breast pocket the handle turns off automatically.

## HEINE BETA handle variants

The handle module (outer sleeve and handle head) is the same for all HEINE BETA handles (with the exception of the large battery handle). The handle type is determined by the bottom insert and the battery type. Deployment of a different HEINE BETA handle is easily achieved by exchanging the bottom insert and/or battery type. An overview of the possible variants is shown on page 3.

## Hygienic Reprocessing

The instruction is available:

- at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

## Maintenance

The device is maintenance free.

## Changing of the Li-ion L battery

For BETA L, BETA 4 NT or BETA 4 USB, remove the bottom inserts as described above. Remove the battery from the bottom insert by pulling it away from the bottom insert and replace it with a new battery. Battery and bottom insert are connected correctly once a distinct click sound has been heard. Screw the bottom insert with the battery back into the handle.

## Service

The device has no components serviceable by the end-user.

## General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you don't use the device for a longer period of time, please remove the batteries in advance.

Ensure that moisture does not penetrate the area around the rechargeable battery and the bottom cap.

Check the rechargeable batteries regularly. In case of any abnormalities in the operating or recharging times, replace the rechargeable batteries.

If the rechargeable battery has not been recharged for a long period it may discharge automatically.

## General Warnings

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform to the safety requirements from IEC 60601-1 Medical electrical equipment.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Turn the handle off during the charging process.

The handle is not designed for continuous use. The operating time of the handle depends on the maximum operating time of the examination head.

### Precautions for HEINE rechargeable batteries

Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.

Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage).

The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

### Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Dispose at your local collection point.

The appendix contains following tables

- Technical specification
- Technical specification HEINE BETA 4 USB
- Technical specifications of the HEINE rechargeable batteries
- Explanation of the used symbols.

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## Technical specification

|   |   |
|---|---|
| Environmental conditions for operation                          | +10 °C to +35 °C<br>30 % to 75 % rel. humidity<br>700 hPa to 1060 hPa |
| Environmental conditions for storage                            | +5 °C to +45 °C<br>45 % to 80 % rel. humidity<br>500 hPa to 1060 hPa  |
| Environmental conditions for transport                          | -20 °C to +50 °C<br>45 % to 80 % rel. humidity<br>500 hPa to 1060 hPa |
| Recommended charging temperature for the rechargeable batteries | +10 °C to +35 °C  |

## Technical specification

### BETA 4 USB rechargeable handle

|                           |                   |
|---------------------------|-------------------|
| Nominal voltage           | 5 V DC            |
| Typical power and current | max. 500 mA       |
| Protection class          | Charging*: I / II |
| Charging current          | approx. 500 mA    |

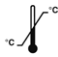
\*) dependant on the USB power supply

## Technical specification of the HEINE rechargeable batteries

|                                     | HEINE Li-ion L<br>(X-007.99.383) | HEINE Li-ion<br>(X-007.99.381) | HEINE NiMH<br>(X-002.99.315) | HEINE mini2Z<br>(X-001.99.487) |
|-------------------------------------|----------------------------------|--------------------------------|------------------------------|--------------------------------|
| Cell type                           | Lithium-ion technology           |                                | Nickel-metal hybrid          |                                |
| Rated voltage                       | 3.7 V                            |                                | 3.6 V                        | 2.4 V                          |
| Capacity                            | 2300 mAh                         |                                | 700 mAh                      | 750 mAh                        |
| Charging time<br>NT300/miniNT       | approx. 2 h                      | approx. 2 h                    |                              |                                |
| Charging time<br>NT4                | approx. 2 h                      | -                              |                              |                                |
| Charging time USB-<br>power adaptor | approx. 4 h                      | -                              |                              |                                |
| Dimensions                          | Ø 26 x 83 mm                     | Ø 26 x 82 mm                   | Ø 26 x 82 mm                 | Ø 14.4 x 107 mm                |
| Weight                              | 62 g                             | 68 g                           | 70 g                         | 37 g                           |

**Erläuterung der verwendeten Symbole**  
**Explanation of utilized symbols**  
**Explication des symboles utilisés**  
**Explicación de los símbolos utilizados**  
**Spiegazione dei simboli utilizzati**  
**Förklaring av symboler som används**  
**Verklaring van de gebruikte symbolen**  
**Forklaring af de anvendte symboler**  
**Symbolforklaring**  
**Käytettyjen symbolien selitys**  
**Explicação dos símbolos utilizados**

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