Instruction for use
HEINE EN200 | EN200-1

Instruction for use

HEINE EN200 | EN200-1

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

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HEINE EN200|EN200-1

These instructions apply to the following products of the HEINE Wall Transformer series: EN200 and EN200-1

 Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE EN200 wall transformer is a dual-channel, dimmable, 3.5 V voltage supply unit with an additional 5 V output for medical diagnostic equipment from HEINE.

The HEINE EN200-1 wall transformer is a single-channel, dimmable, 3.5 V voltage supply unit for medical diagnostic equipment from HEINE. The product can be used either in combination with the EN200 wall transformer or as a stand-alone device.

The product may only be used by qualified medical professionals and in professional healthcare facilities.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Assembly and disassembly

Assembly accessories

Allen wrench 1.3

 To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the switch mode power supply is accessible and disconnectable.

Before mounting the device, check whether you need to use a special wall plug for the subsurface of your wall and whether the subsurface exhibits sufficient load-bearing capacity for the mechanical stress.

 The wall plugs supplied with the product are universal wall plugs and suitable for most building materials (e.g. concrete, solid brick, brick).

1. Hold the drill template horizontally in place on the desired position and mark the drill holes. Drill a hole of at least 40 mm deep using a \varnothing 6 mm drill.
2. Fit wall plugs.
3. Use a suitable screwdriver to tighten the two upper screws with a distance of approx. 3 mm to the wall.
4. Hang the device – disconnected from the mains – on the two screw heads, press firmly and push downwards. Check to make sure that the device is correctly positioned on both screw heads.
5. Screw in the two lower screws.

To disassemble, disconnect the device from the mains first, loosen the two lower screws, push upwards and take off the wall.

USB connector

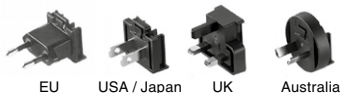
On the back of the EN200 is a USB-A output for the supply of voltage to other medical diagnostic equipment from HEINE.

The EN200-1 has a micro-USB input on the back for the supply of voltage via a medically approved USB power supply or via the EN200.

Connect the HEINE USB cord and run the cable via the housing canal to the outside on the left or right hand side. To do so, press the cable into the clamping hubs, which act as strain relief.

Setting up

Primary plug EN200

Primary plug E4 USB

 The device must only be operated at the mains voltage specified on the type plate.

EN200

- Attach the primary plug to the plug-in transformer until it engages.
- Connect the device to the mains with the plug-in transformer. The green indicator light on the EN200 illuminates.

EN200-1

- Connect the USB cable on the back of the EN200-1 to the plug-in power pack, push the primary adapter onto the plug-in power pack until it engages, and then connect this to the mains supply. The green pilot light on the EN200-1 lights up.

Connecting instruments

The cable handles are fitted with HEINE automatic connectors for use with all similarly equipped HEINE instruments.

- Hold the handle in one hand and insert the instrument connector into the guides in the handle head.
- Push the instrument into the handle until it locks in place with an audible "click".
- Instruments can be locked to prevent unauthorized removal by using the enclosed allen wrench according to fig. 1.
- To unlock, the screw may only be unscrewed so far until the connector can be turned.
- The instrument is ready for operation when taken off the unit.
- To adjust the brightness of the instrument activate the adjustment ring on the handle.
- Both handles can be simultaneously used independent of each other.
- To remove the instrument turn the upper conical ring to the right and pull the instrument off.

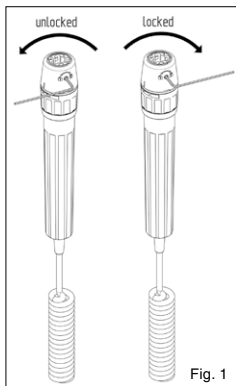
Fig. 1

-  The device enters power saving mode, if the handles are taken off the unit and are not used for more than 15 minutes. The handles are activated by adjusting the brightness or by placing the handle back and taking it off the unit again.

The setting up and operation of the HEINE instrument heads are described in a separate instruction of use.

Hygienic Reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The EN200, EN200-1 and the power supply are maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you are using both devices in combination, the simultaneous operation of three XHL instruments at maximum brightness is not permitted.

General Warnings

 Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the safety requirements from the IEC 60950-1 Information technology or IEC 60601-1 Medical electrical equipment.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

The power supply must not be opened (risk of electrical hazard).

Use power supply in dry environment only. Avoid splashing water. Do not immerse in liquids.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

The device must not be placed near strong Magnetic fields, e.g. MRI units.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the EN200 and EN200-1, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed
Voltage Fluctuations/Flicker	Passed

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 10 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa

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