**Instruction for use**

**HEINE GAMMA®**

**3.1 | 3.2 | 3.3 | C3**

# **Instruction for use**

## **HEINE GAMMA**

### **3.1 | 3.2 | 3.3 | C3**

<b>DEUTSCH</b>	<b>3</b>
<b>ENGLISH</b>	<b>6</b>
<b>FRANÇAIS</b>	<b>9</b>
<b>ESPAÑOL</b>	<b>12</b>
<b>ITALIANO</b>	<b>15</b>
<b>SVENSKA</b>	<b>18</b>
<b>NEDERLANDS</b>	<b>20</b>
<b>SUOMI</b>	<b>23</b>
<b>PORTUGUÊS</b>	<b>26</b>

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 Please read and follow these instructions for use and keep them for future reference.

### Intended use

HEINE GAMMA stethoscopes are used for the auscultation of the activity of human organs (e.g. the heart, lungs, intestines, arteries) and in conjunction with non-invasive blood pressure measurement for the detection of Korotkow sounds.

The product must only be used by qualified medical professionals and in medical healthcare facilities.

### Contraindications

There are no known contraindications to date.

 **For U.S. only:**  
**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

### Warnings and safety information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

### Product overview

The stethoscope consists of chest piece, stethoscope tube and binaurals with separate ear olives. HEINE GAMMA Stethoscopes are available with various chest pieces and tubes. Please see the following description of the different models.

#### **GAMMA 3.1 Pulse Stethoscope**

Especially useful for blood pressure measurement. The chest piece fits comfortably under the cuff of the sphygmomanometer.

#### **GAMMA 3.2, 3.3 Acoustic Stethoscope**

For the auscultation of high and low frequency tones. Switchable.

#### **GAMMA C3 Cardio Stethoscope**

For the auscultation of high and low frequency tones. Switchable.

Special features: dual membrane, bell and double channel tube. The dual membrane permits auscultation of high and low frequency sounds and the bell facilitates evaluation of lower frequencies. (Cardiology)

### Operation

Place the stethoscope on the head so that the eartips are pointing in the line of sight. This ensures that the eartips are inserted into the ear from behind according to the anatomy of the ear canal. Regularly ensure that the eartips are firmly attached to the ends of the headset.

#### *Adjustment of the tension of the binaurals*

Adjust to fit correctly.

#### *- Binaurals are too tight for comfort*

To reduce the spring tension, firmly grasp each section of the binaurals between thumb and fore finger and widen carefully.

#### *- Binaurals are too loose*

To increase the spring tension, grasp both sides of spring with one hand and squeeze them together.

This process can be repeated as often as needed.

### *Switching the double chest piece*

Depending on the needed frequency range, select either the membrane or the open bell side. In order to change hold the connecting tube in one hand and the chest piece in the other, now turn it 180° until there is a noticeable click.

If the connecting tube points towards the membrane: Auscultation with the bell

If the connecting tube points towards the bell: Auscultation with the membrane

### **Hygienic Reprocessing**

The instruction is available:

- at [www.heine.com](http://www.heine.com)

- in a paper version which you can request from the address listed

### **Maintenance**

The device is maintenance free.

### **Service**

The device has no components serviceable by the end-user.

### **General Notes**

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Store and use the device only in a dry and dust-free environment.

Do not dip in cleaning fluid and ensure that no fluids enter the tubing system.

Do not expose the instrument, in particular the soft components, to direct ultraviolet radiation.

HEINE GAMMA stethoscopes are DEHP and latex-free.

### **General Warnings**

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in presence of flammable gases/liquids, or in an oxygen rich environment.

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

To prevent injury:

- The ear pieces should be firmly attached to the binaurals.

- Do not insert the ear pieces too far into ear canal, check for cleanliness regularly and clean as needed.

- Never use a stethoscope without ear pieces.

- Never use a stethoscope without a membrane.

Measurements should only be interpreted by qualified persons.

Avoid unnecessary impact on and vibration of the instrument.

Avoid damaging membrane and soft components with sharp or pointed objects.

**Disposal**

Disinfect the product before disposal. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Technical specification
- Spare parts
- Explanation of the used symbols

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